

#### Device description

The Magic<sup>3</sup> Go<sup>®</sup> Intermittent urinary catheter is a silicone tube inserted into the urethra for the purpose of draining the bladder of urine.

Not made with natural rubber latex. Does not contain DEHP.

#### Indications for use

The Magic<sup>3</sup> Go<sup>®</sup> Intermittent urinary catheter is intended for urological use only. It is intended for use by adult and pediatric patients of all ages for bladder management including urine drainage, collection, and measurement. The device is passed to the urinary bladder via the urethra.

#### Contraindications

None known

#### Warnings

This is a single use device. Do not resterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.


#### Precautions

Inspect the catheter before use. Do not use the product if the device or packaging is damaged. Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided

#### Adverse Reactions

Urinary Tract Infection  
Bleeding from the urethra  
Irritation of the urethra

Please consult product inserts and labels for any indications, contraindications, hazards, warnings, cautions and directions for use.

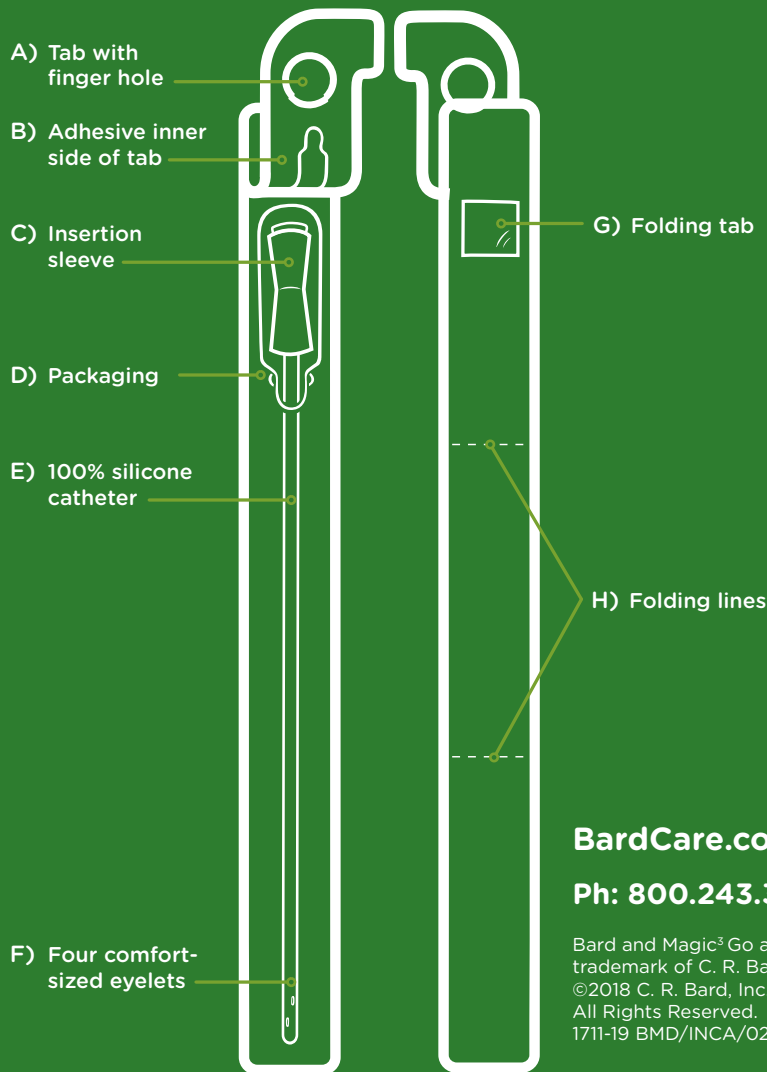

**BardCare.com**

**Ph: 800.243.3315**

Bard and Magic<sup>3</sup> Go are registered trademark of C. R. Bard, Inc.  
©2018 C. R. Bard, Inc.  
All Rights Reserved.  
1711-19 BMD/INCA/0218/0207

# Magic<sup>3</sup> Go<sup>®</sup>

HYDROPHILIC INTERMITTENT CATHETER  
with CleanGlide<sup>™</sup> Technology

## INSTRUCTIONS FOR USE

For use with 16" male length catheters  
and 10" pediatric length catheters



## Instructions for use

Review the self-catheterization procedure with a healthcare professional.

1. Always wash hands prior to use.
2. Open the catheter package by peeling the tab upwards with the aid of the finger hole.
3. If desired, use the adhesive label(s) to hang the package on a nearby dry vertical surface while preparing to catheterize.
4. Clean the opening of the urethra—and the surrounding area. Wash your hands again.
5. Hold the insertion sleeve with your dominant hand and squeeze it to grip the catheter shaft as you remove the catheter from the pack. Next, hold the catheter funnel above the insertion sleeve with your other hand and slide down the shaft, stopping at about six inches from the tip. Release the funnel. Using the insertion sleeve to hold the catheter firmly, gently pass the tip of the catheter firmly into your urethra until the insertion sleeve nears the meatus. Repeat until urine starts to flow.
6. Keep the catheter steady until urine stops flowing. When urine stops flowing, slowly withdraw the catheter, stopping if flow starts again, until the last few drops have drained. Check the color, odor and clarity of the urine. Any changes may need to be reported to a health care professional.

### Disposal

7. Place the catheter back into the package and dispose in accordance with accepted medical practice and applicable local, state and federal laws and regulations. Do not flush down the toilet.
8. Wash hands.

